Before using this product, please carefully read and follow the recommendations of the instruction manual originally delivered with your device and system. Please pay special attention to the safety precautions.

Product Description

The AIR-FLOW® Perio (prophylaxis powder) is used for subgingival treatments.

This powder is for use in dentistry with the EMS air polishing devices.

Applications

This powder removes subgingival biofilm and is recommended for the following procedures:

- Removal of periodontal biofilm,
 - Maintenance in periodontal therapy after completion of initial treatment,
 - Maintenance of implant (including polished titanium),
- This powder can be used for standard supragingival applications as described in the AIR-FLOW® system manual and is specially suited for sensitive supragingival applications like:

- Coloration around sulcus,
- Treatment in presence of hidden composites,
- Cleaning of sensitive areas.

Usage

For perio pockets more than 5mm (up to 9mm) use the perio flow nozzle for a maximum of 5 seconds.

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For shallow peri pockets (up to 5mm), use the standard AIR-FLOW® handpiece to spray each side of the tooth (vestibular, mesial, oral distal, lingual) for a maximum of 5 seconds per pocket.

• Do not concentrate the spray on one point; use a circular motion to achieve homogeneous cleaning.

• The angle of the handpiece tip to the tooth can be varied between 30° and 60°. The smaller the angle is, the further the powder will spray into the pocket.

Contra-indications

• Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

• Predisposed persons may be sensitive to glycolic. If allergic reactions are observed, stop using the product and completely remove the powder from mouth.

• The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients (general weakened immunity, endocarditis).

• Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

Recommendations for the treatment

• Patient and user have to wear eye protection.

• User has to wear a mask and gloves.

• Use the high-speed evacuator of your dental unit.

• Calculi and tartar cannot be removed with this powder.

• Apply Vaseline® on the patient's lips.

• Wash hands after use.

• Wash patient face after use (e.g. with a wet wipe).

General information

• Do not exceed the powder filling limit as indicated on the powder chamber.

• This powder is sugar free.

Storage and shelf-life

• The powder should not be stored near heat sources.

• Storage condition -10°C to 25°C and 10% to 95% relative humidity.

• Do not use after expiration date.

Also available:

• AIR-FLOW® CLASSIC available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral (for supragingival treatments).

• AIR-FLOW® CLASSIC COMFORT (for supragingival treatments).

• AIR-FLOW® SOFT (salt free for supragingival treatments).

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